

MRS MOBILE ROOM STERILIZERS SPECS & PRICES



Model **MRS3684P**

These mobile 8-lamp UV sterilizers feature 2-minute "ON" delay and 24-hour timer for total room irradiation. They are designed for operating rooms, sterile areas, laboratories, unoccupied patient rooms, clean rooms and all other applications where permanent fixtures are not practical. Each unit is equipped with casters for maximum portability.

Dimensions - 20"L x 20"W x 48"H

Installation Instructions:

These units are shipped assembled and wired for 120V, 60Hz operation. The only procedure necessary is installation of the eight 4-pin **GML100** germicidal lamps.

To install the lamps remove the top thumb screw and slide the lamp into 4-pin connector. Use gloves when touching the lamp. If fingerprints get on the lamp, clean the lamp with denatured alcohol prior to operation.

General Operation:

1. Position unit in center of area to be exposed to UV rays
2. Plug unit into 120V, 60Hz outlet
3. Set 24-hour timer for desired exposure time
4. Push "On" button - 2 minute delay switch
5. Leave area or utilize protective glasses. Avoid exposure to direct or reflected UV light
6. Check and clean lamps regularly - annual replacement required if used continuously - 12,000 hours rated life.

Model	Description
	Mobile Room sterilizer with 24 hour timer, 2-minute "ON" delay, casters, and handle (20"L x 20"W x 48"H)
MRS3684P	Eight GML100 lamps 120V, 60Hz
MRS3684P/220	Eight GML100 lamps 220V, 50/60Hz
MRS3684P/SS	Stainless steel Mobile unit with 8 GML100 lamps 120V
GML100	Replacement lamp
TXG015B	Replacement ballast

Warning: Care should be taken to insure that personnel are not exposed to direct or reflected UV light. Suitable eye and skin protection should be employed when lamp is in operation. Before cleaning or relamping, always turn the power OFF.